

Ilonion Care Oy

HMT-1 QUICK GUIDE

- ▶ Press on the HMT-1 from ON/OFF button approx. by 3 seconds.
- ▶ Adjust the headband straps so that the device stays firmly in your head.
- ▶ Adjust the mini monitor so that it does not disturb your vision or that it is not visible in the monitor or in the pictures/videos you take.
- ▶ Give the voice command **"My Camera"**.
- ▶ Place an index finger on the estimated operation area, which should appear as centre as possible to the screen. Adjust accordingly.
- ▶ Zooming: There are 5 different zooming levels available from 1 to 5. Voice command for zooming is e.g. **"Zoom Level 3"**
- ▶ Taking pictures: Voice command is **"Take Photo"**
- ▶ Recording videos: Voice commands are **"Start Video"** and **"Stop Recording"**.
- ▶ For changing the zoom level during operation: **"Stop Recording"** → select desired zoom level e.g. **"Zoom Level 5"** → **"Start Video"**
- ▶ **"Navigate Back"** command brings you always back to the previous stage.
- ▶ **"Navigate Home"** command brings you always back to the home screen.

More detailed instructions and functions are provided in larger user guide.